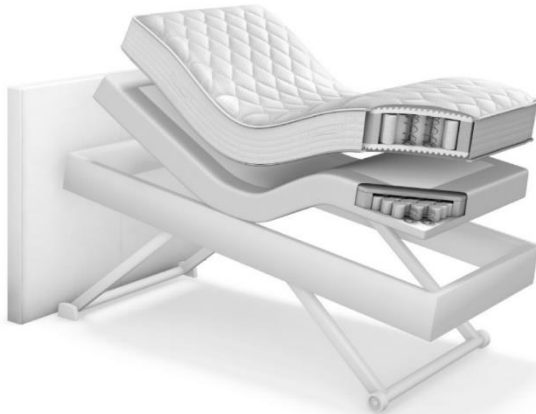
DREAM  
BIG.

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To avoid damaging your upholstered bed, do not open the package with a box knife.

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Do not stand on the corners or allow to fall.

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Any damage resulting from non-compliance with the above instructions will lead to loss of warranty rights.

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<b>Introduction .....</b>	<b>55</b>
<b>1. Intended use .....</b>	<b>57</b>
<b>2. Safety instructions .....</b>	<b>57</b>
2.1. General safety instructions .....	57
2.2. Permitted loads .....	59
2.3. Ambient conditions .....	59
2.4. Moving the bed .....	60
2.5. Repairs, spare parts, accessories .....	60
<b>3. Assembly and operating manual .....</b>	<b>61</b>
3.1. Assembly instructions for the EOX Relax Boxspring .....	61
3.2. Electrical functions .....	63
3.3. Transport position .....	67
3.4. Reset .....	68
<b>4. Accessories .....</b>	<b>69</b>
4.1. Mattress holder .....	69
4.2. Accessory holder .....	70
<b>5. Technical information .....</b>	<b>76</b>
5.1. Technical data – bed mechanism .....	76
5.2. Dimensions and adjustment ranges .....	77
5.3. Technical data – electronic components .....	77
5.4. Higher level mains disconnection .....	80
5.5. Electrical installation diagram for EOS Relax .....	81
5.6. Preventive maintenance and servicing .....	82
5.7. Care instructions for technical components .....	83
<b>6. Instructions on use and set-up of the EOS Relax .....</b>	<b>84</b>
6.1. Noise generation/creaking .....	84
6.2. Indoor climate .....	84

6.3.	Signs of wear.....	85
6.4.	Discoloration from contact with clothing or bed linen, pilling .....	85
6.5.	Colour variation .....	85
6.6.	Finishing features .....	86
6.7.	Odours when new, changes during use.....	86
6.8.	Care and stain removal .....	86
<b>7.</b>	<b>Conformity .....</b>	<b>89</b>
7.1.	Explanation of the CE label and identification label .....	90
7.2.	Declaration of conformity .....	92
<b>8.</b>	<b>Disposal and the environment.....</b>	<b>93</b>
<b>Annex 1 – Maintenance and servicing checklist .....</b>		<b>94</b>
<b>Annex 2 – Symbols used.....</b>		<b>95</b>

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## Introduction

Dear customer,

Thank you for placing your trust in our company by choosing a Philrouge product from DOC.

Before our products leave our factory, they are carefully checked and subjected to a comprehensive quality control process. However, should you experience a problem or have any questions about the product, please contact your local dealer.

To ensure you can continue enjoying your Boxspring bed for many years to come, and to ensure its trouble-free functioning, please carefully read and follow the instructions in this manual.

DOC cannot be held liable for any damage to the bed or any consequences for the user resulting from improper use and/or incorrect care, maintenance or servicing.

DOC reserves the right to make technical changes to the product as well as to this operating manual.

This manual will help you during assembly and use of your new bed. We strongly recommend that you refer to the manual during assembly.

**Attention:** Please refer to the **additional documentation** enclosed with the respective accessories, as this may affect the steps described in this document. Damage to the bed due to failure to observe the various assembly steps described in this manual will void the warranty.

The original version of this operating manual is written in German.

**Please read these instructions carefully and keep them for future reference!**

# 1. Intended use

The EOS Relax Boxspring bed is intended exclusively for **private** use as a base for sleeping, resting and recreation.

The bed is equipped with extensive adjustment options and comfort settings, which also make the bed suitable for use as a nursing bed within application area 4 in accordance with IEC 60601-2-52:2009, namely:

To alleviate or compensate for an injury, disability or illness in home care.

The bed is classified in medical risk class 1 as a non-invasive medical product in accordance with Medical Directive 93/42/EEC.

The bed is not suitable for people under 146 cm in height and with a body mass index  $\leq 17$ .

All electronic components are splash-proof in accordance with IPX 4.

The bed must not be used in humid or wet rooms, or in environments in which flammable or explosive gases or vapours may be generated or present.

**To take advantage of all the bed's features, it is necessary to use a moveable and flexible mattress to ensure maximum functionality and comfort.**

## 2. Safety instructions

### 2.1. General safety instructions

The manufacturer, importer or distributor of this product can only be held responsible for its safety, quality and reliability if the bed is installed, maintained and used in accordance with the following instructions for assembly, operation and use.

Do not allow children to play on or near the bed – children may be unaware of the potential hazards associated with electrically adjustable beds.

The bed system is likewise unsuitable for adults with certain types of disability, who may require supervision and/or thorough instruction in its use. Especially when retracting the bed, make sure that no objects or limbs are between the moving parts or under the bed, as this may result in serious crushing injuries.

The bed may only be connected to 230V / 50Hz alternating current, with a minimum socket fuse of 6 or 10V. The voltage must match the type plate on the appliance. Use sockets with an upstream residual current circuit breaker.

Never use defective power, motor or manual-switch cables. Replace defective cables immediately. Do not pull or clamp the power cable around corners or sharp edges. Do not lay or hang the power cable above hot objects. Never pull the plug out of the socket by the power cable or with wet hands.

Only use electrical devices that are in perfect working order in the vicinity of the bed. Ensure that these devices are never connected to multiple sockets under the bed (risk of electric shock and fire resulting from liquid ingress). Do not use extension cords and multiple sockets. Ensure that the power cables of auxiliary equipment (reading lamps, etc.) cannot be pinched or damaged when the bed is adjusted.

Also ensure that the bed's range of movement is not restricted by people (especially playing children) or other obstacles.

Lay the mattress, blanket and/or sheets on the bed in a way that ensures they do not impair the bed's functions.

All parts must be free to move. Do not place the bed near window sills or other obstacles.

Do not allow people other than the user to sit on the bed.

To increase the user's safety, we recommend placing the bed in its lowest position, especially when sleeping.

## **2.2. Permitted loads**

Maximum safe working load (including mattress, user, attachable accessories)	200 kg
Maximum occupant/patient weight	150 kg
Maximum load on the backrest and foot section (each)	80 kg

The maximum safe working load requires a weight distribution of approx. 45% on the backrest, 25% on the middle section and 30% on the foot section.

## **2.3. Ambient conditions**

Equipment that generates strong magnetic fields which could interfere with the bed's control unit should not be used in the bed's immediate vicinity. Conversely, the bed may generate electromagnetic fields during operation that may affect nearby measuring equipment; nevertheless the bed complies with the EN 60601-1-2 standard. Some residual risks remain.

The bed must not be used if there is a risk of explosion or in the presence of flammable, volatile gases, e.g. anaesthetics.

The bed may only be used under the following ambient conditions:

- Temperature: between 5°C and 40°C
- Humidity: 20% to 90% at 30°C without condensation
- Air pressure: 700 to 1060 hPa

In case of higher temperatures during delivery, let the bed rest for 24 hours before using it or connecting it to the mains.

## 2.4. Moving the bed

The bed is equipped with a travel unit.

- Always disconnect the mains plug before moving the bed.
- Never move the bed over – or pull on – the power cable.
- Ensure that the bed is not resting on the power cable when in the parked or starting position, and that the cable is not caught between the moving parts.

Adjusting the headboard and footboard is **not permitted** as long as the bed is in the transport position.

## 2.5. Repairs, spare parts, accessories

In case of faults or suspected defects, unplug the power cable immediately.

The bed must only be mechanically or electrically repaired by trained personnel. We cannot accept any liability in the case of improperly executed repairs. The housings of drives and controls, etc. must not be opened.

DOC cannot be held liable for any damage or injuries resulting from unauthorised modifications and/or use of non-original spare parts or accessories. Technical changes may only be carried out with the manufacturer's written approval.

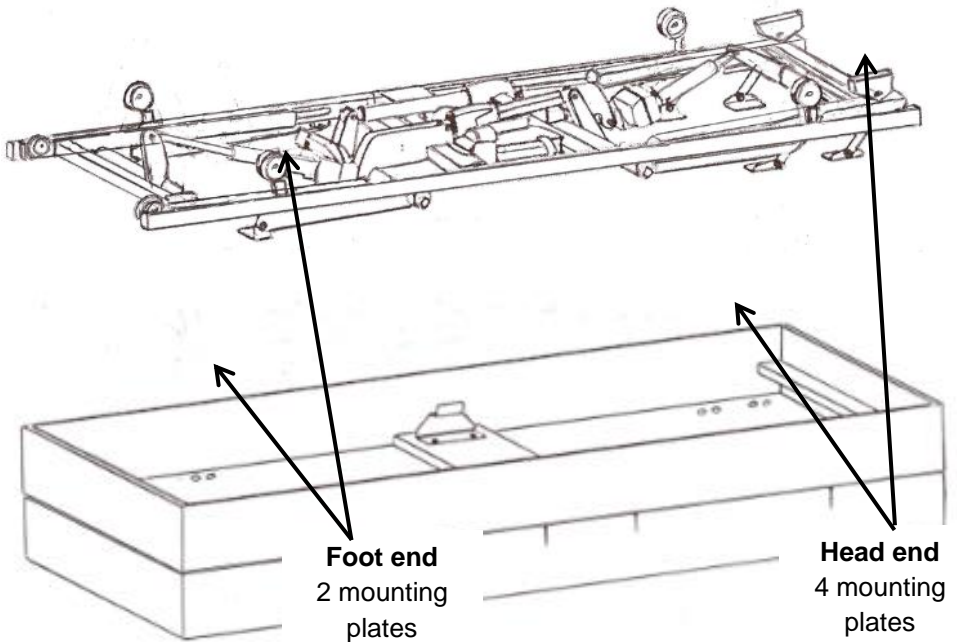
Only use original spare parts and accessories!


### 3. Assembly and operating manual

#### 3.1. Assembly instructions for the EOX Relax Boxspring

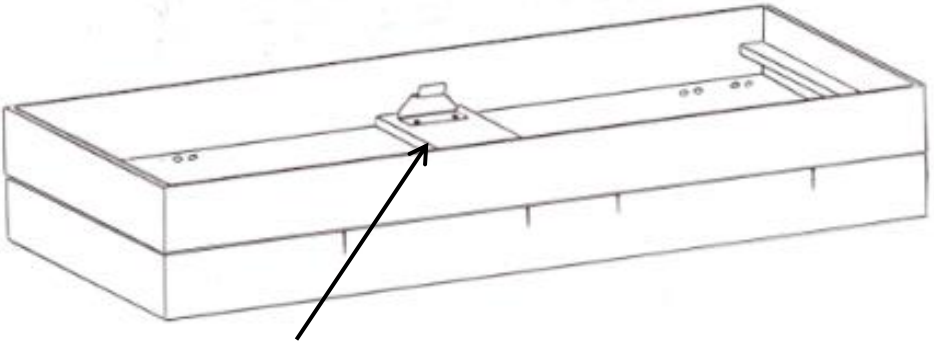Carefully unpack the metal frame and the padded box and place them side by side so that the head and foot ends of the metal frame and box are facing in the same direction.




If not already pre-assembled, insert the two metal brackets in the centre.

	ISO 7380- M8x20	4x
	● n°5	

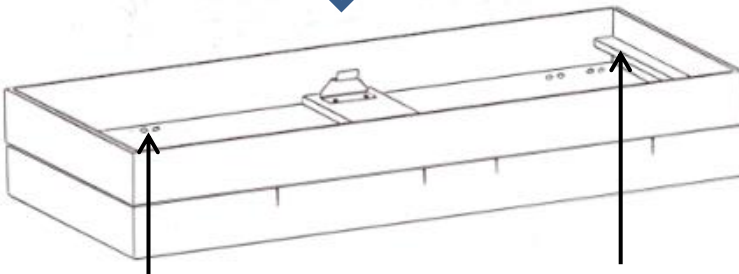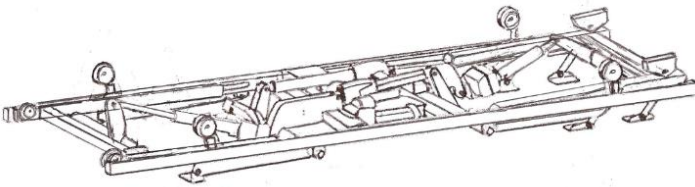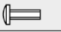Then insert the metal frame into the padded box.

	ISO 7380- M6x25	12x
	● n°4	


	ISO 7380-I M8x60	4x
	● n°5	

Tighten the 4 screws on the outer frame slightly, so that the frame is still movable. Insert the screws for the mounting plates (4 plates at the head end, 2 plates at the foot end). Then tighten all the screws.

Turn the box and place it on the metal frame.

### 3.2. Electrical functions

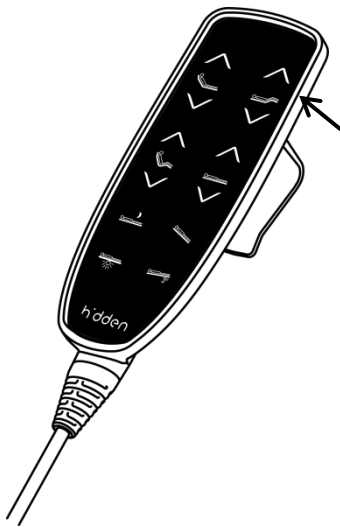Please explain the electrical functions to the user and inform him or her of any possible risks when using the bed. Only users with sufficient knowledge and experience of using the product may operate the bed. Users with disabilities who require supervision may only operate the bed under adequate supervision.

Soft-touch buttons

If two function keys are pressed simultaneously, the controller reports a fault and the bed stops.

To ensure the user's safety, make sure that the handheld transmitter is kept out of reach of the user, and especially of children. This applies in the following cases:

- If children are left unsupervised near the bed.
- If a particular bed function could be hazardous for the user.
- If the user is unable to operate the bed safely or if they may endanger themselves while operating the bed.
- If additional equipment or accessories create a potential hazard for the user.

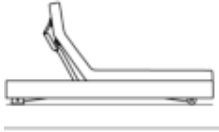
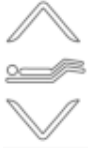
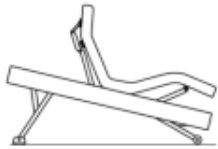
Never operate the motors for more than 2 minutes without interruption. Subsequently, a pause of 18 minutes is necessary before the bed can be adjusted again.

The integrated handheld transmitter may interfere with implanted, active medical devices, such as pacemakers, defibrillators, etc. It is the responsibility of the user/operator to take appropriate precautions to avoid any risks.


## Adjustment options

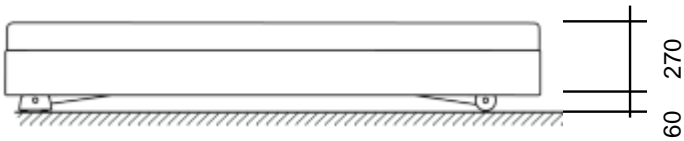	Setting	Handheld transmitter	
Backrest			Maximum adjustment angle = 55° with simultaneous headboard adjustment
Foot section			Maximum adjustment angle = 35°
Relax position			The backrest and headboard are moved simultaneously. To adjust only the backrest, see the anti-Trendelenburg adjustment
Up / down position			Between 250 and 420 mm, the bed only moves horizontally to avoid collisions with the floor

	Setting	Handheld	
Sleep position			All sections are positioned horizontally. The bed stops at a height of approx. 12 cm. Press the button again for the lowest position.
Anti-Trendelenburg			Highest position for full body elevation = 13 °
Legs-up position			= Trendelenburg position Foot elevation 10 cm higher than the head
Transport			
Floor lighting			


Sleep position


### 3.3. Transport position

To move the bed safely, the following steps are necessary:

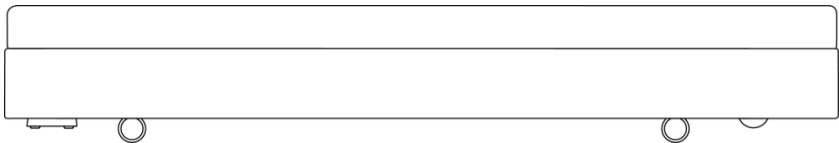- Adjust the bed to the transport position.
- Never move the bed unless it has been disconnected from the mains. Never pull on the power cable.
- If the power cable is not carefully stowed during transport, the cable may be damaged, creating a significant risk of electric shock. Make sure that the power cable is not torn, crushed or squeezed. Do not move the bed over the power cable.


Relax transport position

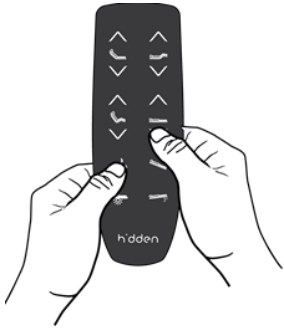Press and hold the “Sleep” and “Up/Down” function buttons simultaneously until the bed is resting on the castors.


### 3.4. Reset

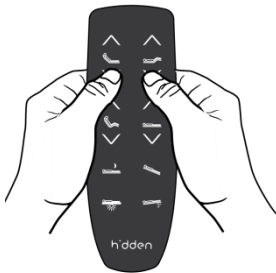A controller fault is indicated by an audible beep when a button on the handheld transmitter is pressed. The fault can be resolved by resetting the handheld transmitter and the controller:

Simultaneously press both buttons on the second row of controls.

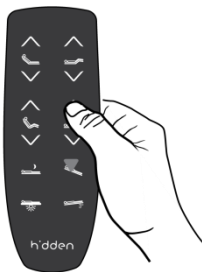An audible beep confirms the start of the reset procedure. Hold down both buttons until the beep is no longer heard.

Then move the bed to the highest position; the controller will then reinitialise itself.


## 4. Accessories

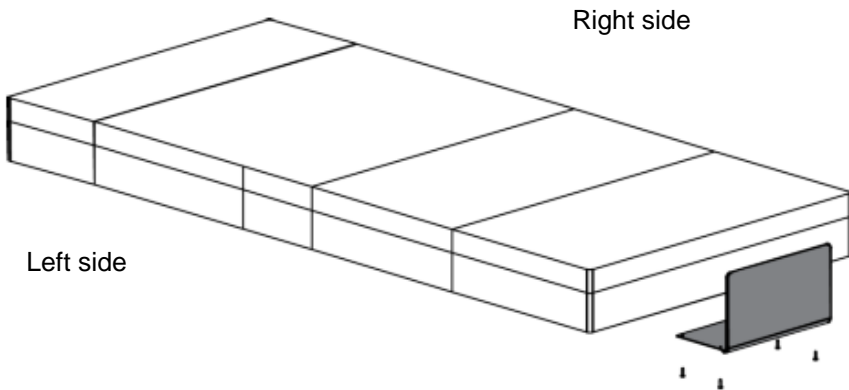Your EOS Relax Boxspring can be supplemented with the following accessories:

- Mattress holder
- Accessory holder
- Trapeze bar
- IV pole

### 4.1. Mattress holder

Your EOS Relax Boxspring is equipped with an additional mattress holder at the foot end. The mattress holder is enclosed in the padded box.


	ISO 7380- M8x25	4x
	● n°5	

## 4.2. Accessory holder

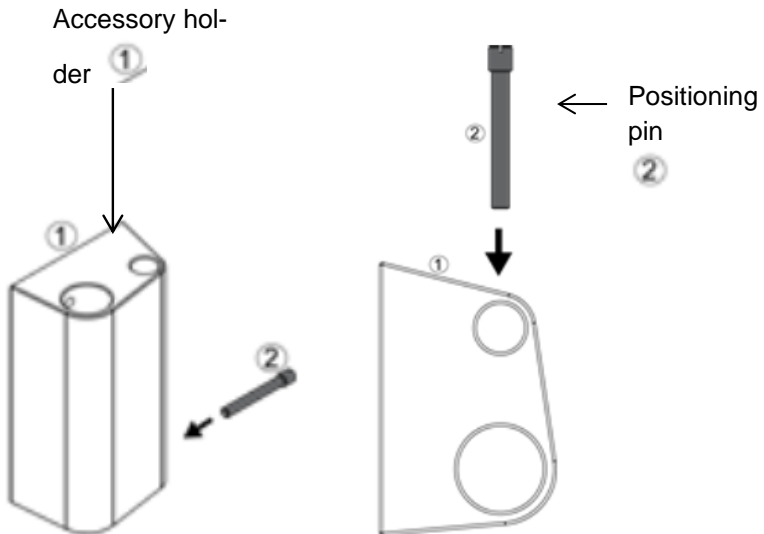
Can be mounted on both the right and left side of the bed at the head end. The holder is designed to mount the trapeze bar and IV pole.

First, the positioning pin must be used to limit the tilt angle of the trapeze bar. The position of the positioning pin depends on whether the accessory holder is mounted to the left or right side of the bed. The left and right sides of the bed are defined as shown in Figure 4.1.

### 4.2.1. Preparing the accessory holder for the right side:

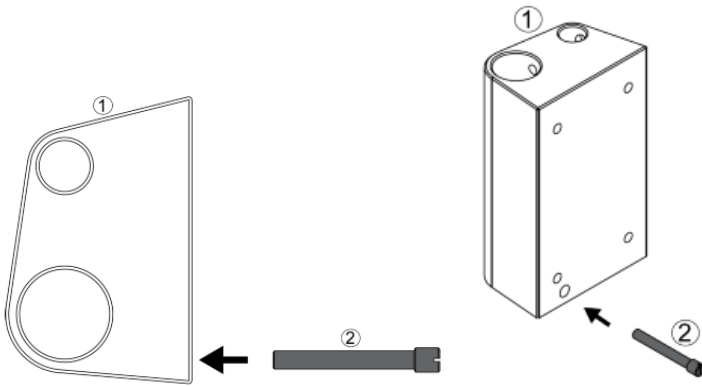
4.2.2. Preparing the accessory holder for mounting on the left side:


The accessory holder is screwed through the side of the bed. There are pre-drilled holes on both sides of the bed. Use a sharp needle to pierce the padding at the location of the pilot holes.

On the inside of the box, the locations of the pilot holes are indicated by square markings with staples.

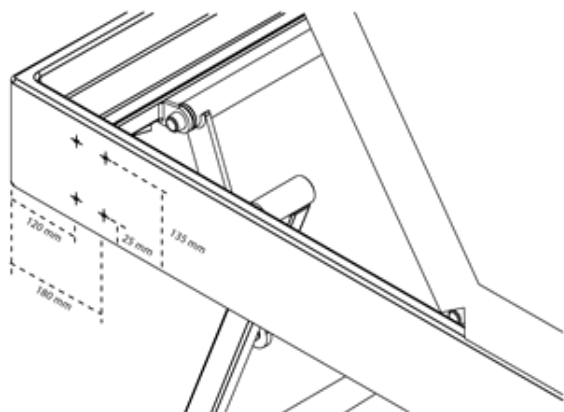Pilot holes on  
the left side of  
the bed





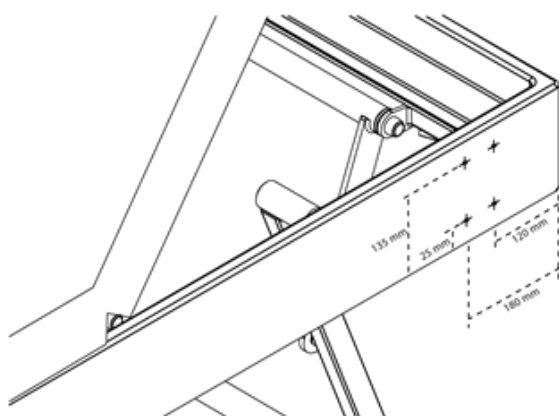
Pilot holes on the  
right side of the  
bed


### 4.2.3. Mounting the accessory holder
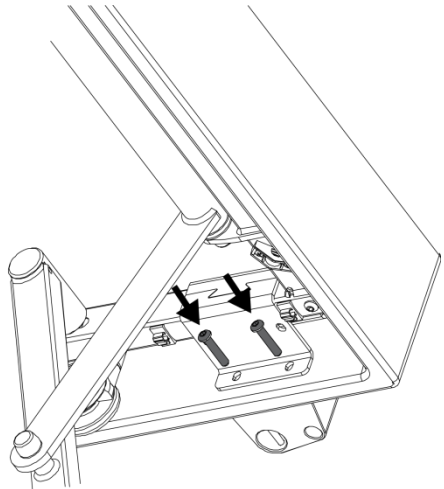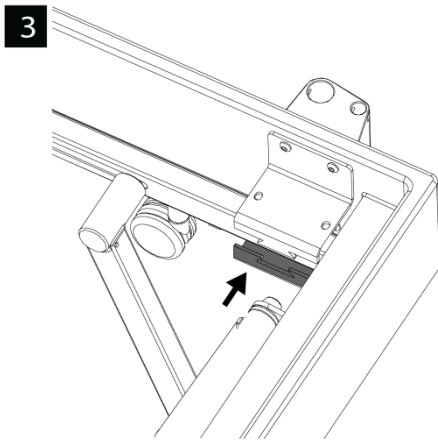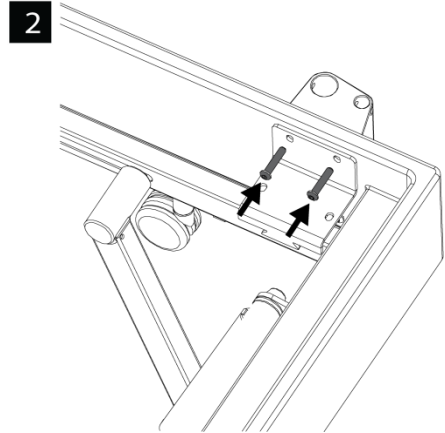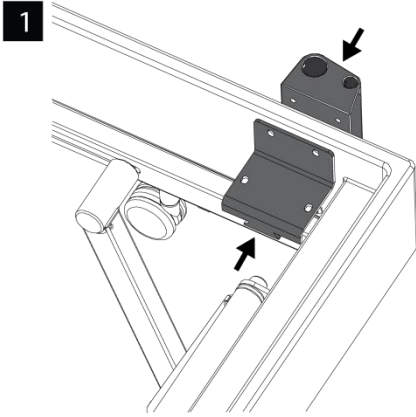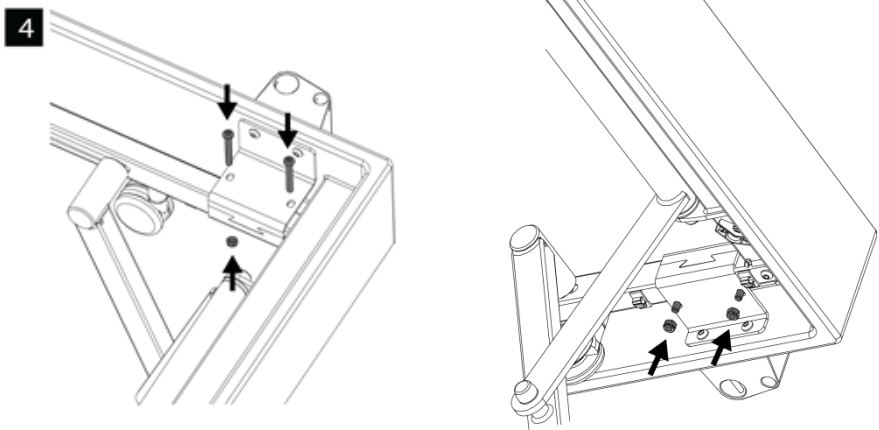
#### 4.2.4. Trapeze bar

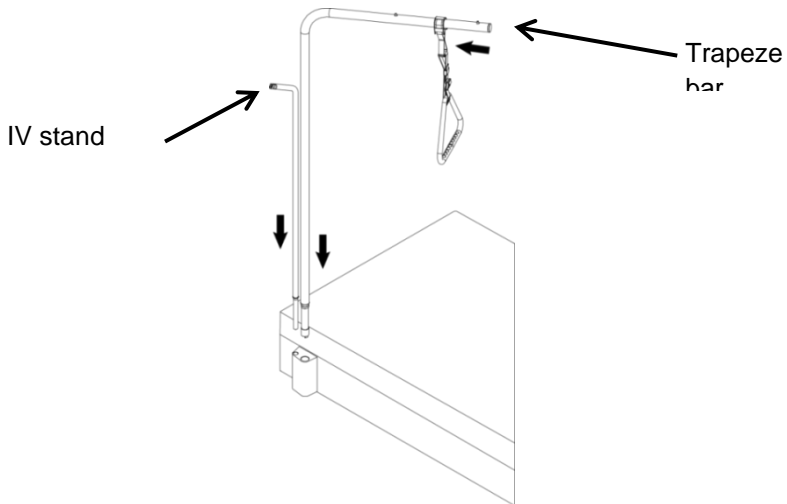The trapeze bar may only be attached to the designated accessory holder. Make sure that the trapeze bar is inserted fully into the accessory holder.


The trapeze bar can be rotated 90°, however it cannot swing out beyond the side of the bed.

Always ensure a sufficient safety reserve for the load – the maximum static load is 75 kg:

**The trapeze bar is not suitable for exercises for rehabilitation purposes!**

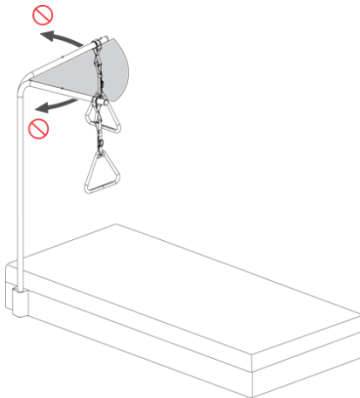It is advisable to inspect the handle of the trapeze bar regularly and to replace it every 4 to 5 years.

Outer movement  
limits of the tra-  
peze bar

Weight: 4.9 kg

Max. static load: 75 kg

**If the bed is equipped with a trapeze bar and/or an IV stand, carefully check the surrounding areas, especially if the bed is adjusted or moved. This will prevent damage to the bedding and accessories. Always make sure that no one can get trapped in – or by – the bed.**

#### 4.2.5. IV stand

The infusion stand can be rotated 360°. It is equipped with a hook for attaching infusion bottles.

Weight:	0.75 kg
Max. load on the hook:	4 kg
Max. load on the IV stand:	8 kg

## 5. Technical information

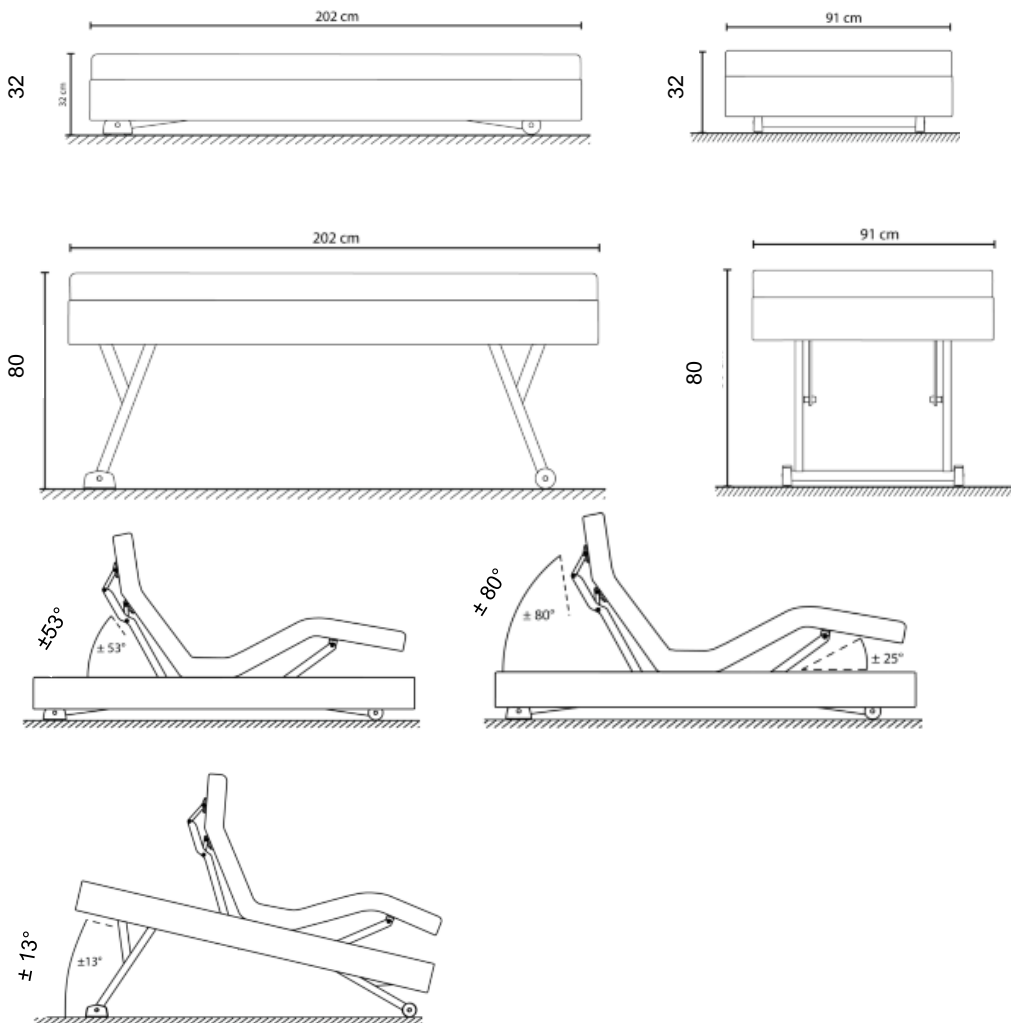
### 5.1. Technical data – bed mechanism

This data is valid for bed sizes 90 x 200 cm and 100 x 200 cm

Empty weight of the bed without accessories	120 kg
Weight of lifting mechanism	60 kg
Weight of box with motorised frame	60 kg
Max. safe working load (including mattress; accessories; user)	200 kg
Max. weight of user	150 kg
Max. load on backrest and foot section during adjustment	80 kg
Height of user	≥ 146 cm
Body Mass Index (BMI)	≥ 17
Electrical protection level (splash-proof)	IP X4

Please note that the maximum safe working load can only be applied if the bed is loaded in accordance with EN 60601-2-52 (backrest 45%, middle section 25%, thigh and foot section 30%).

## 5.2. Dimensions and adjustment ranges

## 5.3. Technical data – electronic components

All electronic components are protected at least in accordance with IPX 4 protection (against splashing water).

### 5.3.1. Controller

Type	C061
Manufacturer	Linak
Made in	Denmark
Electrical protection type	IPX6
Primary supply line / max. current consumption	230 VAC / 1.25 A
Frequency	50 – 60 Hz
Secondary current	24 V
Load	max 10% or 2 min/18 min
Length of power cable	3 m
Plug	welded Euro plug
Standby mode	0.3 W

### 5.3.2. Manual switch

Type	HB 80
Manufacturer	Linak
Made in	Denmark
Electrical protection type	IP66
Max. current consumption	30 mA

### 5.3.3. Backrest motor

Type	LA 27 C
Manufacturer	Linak
Made in	Denmark
Electrical protection type	IP X4
Input voltage	24 DC
Stroke length	200 mm
Speed	7 – 11mm/s
Noise emission	max 45 dB(A) DS/EN ISO 3746
Load duration	max. 10 % of 2 min / 18 min
Thrust	3500 N

#### 5.3.4. Foot section – motor

Type	LA 27 C
Manufacturer	Linak
Made in	Denmark
Electrical protection type	IP X4
Input voltage	24 DC
Stroke length	60 mm
Speed	3.6 – 6 mm/s
Noise emission	max 45 dB(A) DS/EN ISO 3746
Load duration	max. 10 % of 2 min / 18 min
Thrust	6000 N

#### 5.3.5. Up/down motor

Type	LA 40
Manufacturer	Linak
Made in	Denmark
Electrical protection type	IP X6
Input voltage	24 DC
Stroke length	250 mm
Speed	3.6 – 6 mm/s
Noise emission	max 45 dB(A) DS/EN ISO 3746
Load duration	max. 10 % of 2 min / 18 min
Thrust	8000 N

**Never adjust the bed continuously for more than 2 minutes. In order to protect the motor against overload, wait at least 18 minutes before further adjustment.**

#### **5.4. Higher level mains disconnection**

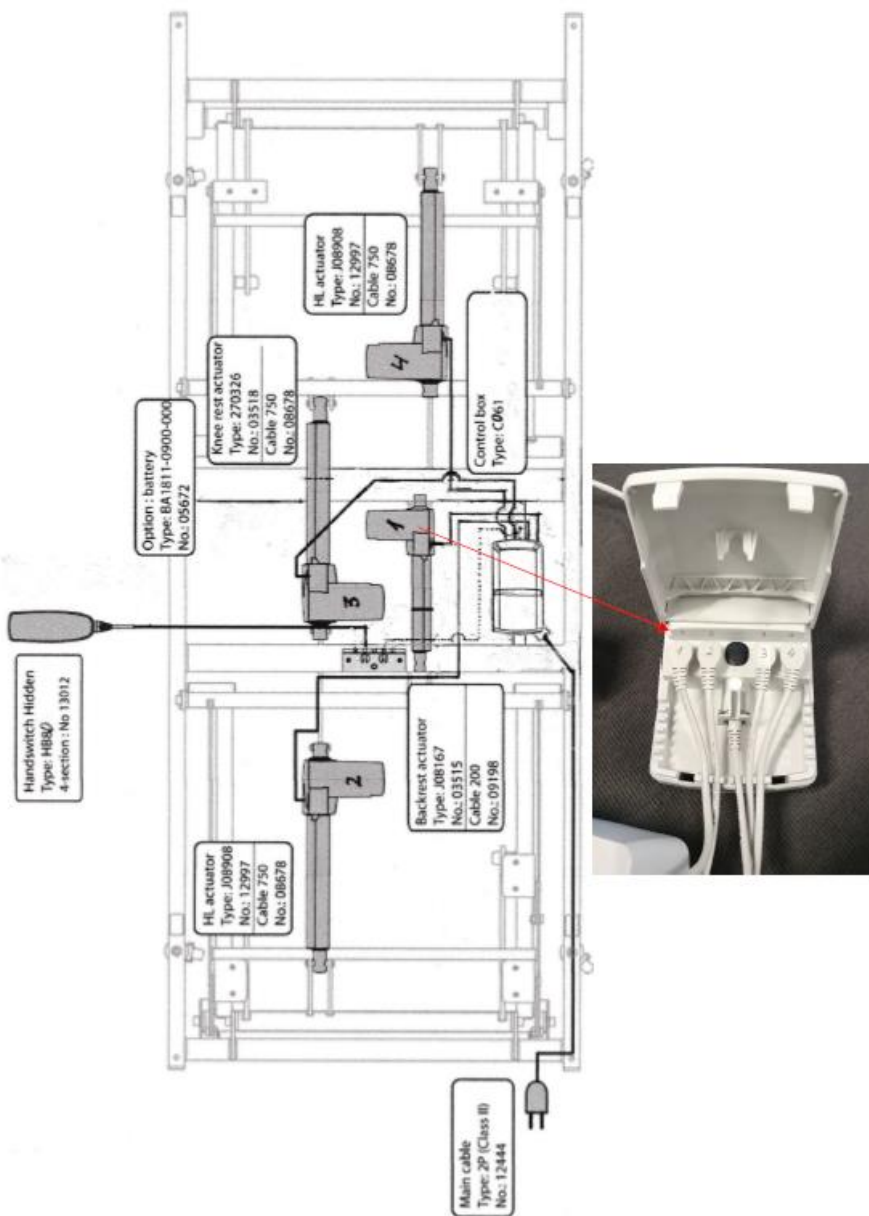
The motor system is not compatible with higher-level household mains disconnection. Possible solutions:

- a) As a preliminary step, have the socket wired separately by an in-house electrician with the wiring bypassing the household mains disconnection.
- b) Installation or interposition of a switchable socket or terminal strip; only for use outside the bed.

This ensures that your bed's motor system can be completely separated from the mains supply, thus ensuring the functioning of the household mains disconnection.

## 5.5. Electrical installation diagram for EOS Relax

The C061 controller is equipped with a safety device that detects a so-called “first fault” as a result of the controller checking the hardware.

If the system is working properly, the LED on the controller will change from green (standby mode) to yellow as soon as a button on the handheld transmitter is pressed. If the LED changes to yellow without a button being pressed, a “first fault” has been detected.

Nevertheless, the controller may also continue to work without problems after a first fault occurs, i.e. when the LED lights up yellow. The product will remain safe to use until the next scheduled service interval. The cause of the fault must then be remedied during the next inspection.

The safest way to determine the cause of the problem is to disconnect one cable at a time from the controller. When the LED changes back to green, you have identified the defective element that needs replacement. If the LED remains yellow after all the cables have been removed, the controller itself must be replaced.

The LED is located on the controller and is visible from above when the backrest is raised.

The controller is protected by an internal fuse.

**Never replace a cable or motor when the controller is connected to the mains and/or a function is being carried out!**

1. Disconnect the controller from the mains and wait 5 seconds.
2. Replace/install the cable and/or motor.
3. Check all cable connections; the rubber O-rings must be present. Install the cable and reattach it with suitable fastenings.
4. Connect the controller to the mains and carry out a function test.
5. Unused connections must be sealed with caps to ensure the intended degree of electrical protection.

## **5.6. Preventive maintenance and servicing**

To ensure the user's safety throughout the product's service life, we recommend having the bed checked and serviced once per year. **If the bed is used**

**in private households, this inspection is not mandatory; however it is required by law in public areas.**

All steel components are protected by powder coating or chrome plating. The pivot points are self-lubricating and maintenance-free.

**Maintenance tasks may only be carried out by qualified and technically adept personnel. Improper maintenance or repairs that cause damage to the bed will void the warranty.**

**All repairs to the motors, controller and motor accessories must be performed by Linak, an authorised Linak service centre or a service technician trained by Linak. The opening of any Linak components will void the warranty.**

**The bed may not be modified without the express written consent of DOC AG. After an authorised modification, the bed should be subjected to a thorough inspection and tested for functionality and safety.**

**Never carry out any repairs or maintenance on the bed while it is being used.**

**For the annual inspection, use the checklist in the Annex to this operating manual.**

## **5.7. Care instructions for technical components**

### **Motors/controller**

Before cleaning, disconnect the mains plug. The motor/controller should only be cleaned with a dry or slightly moistened cloth. Do not use solvents (petrol, alcohol, etc.) and avoid ingress of liquids. Do not use household detergents. If motors or push rods are heavily soiled or insufficiently coated by a greasy film, clean them with a dry cloth and re-grease with Teflon (PTFE) spray.

### **Manual switch**

The manual switch is maintenance-free. Clean the housing with a slightly moistened cloth. Do not use aggressive detergents or abrasives, or petrol,

alcohol, etc.

### **Lift mechanism and metal components**

Clean only with a dry or slightly damp cloth. Do not use solvents (petrol, alcohol, etc.) and avoid ingress of liquids. Do not use household detergents.

## **6. Instructions on use and set-up of the EOS Relax**

### **6.1. Noise generation/creaking**

Our bed systems are designed and constructed to prevent noise or keep it to a minimum, so that it does not become a nuisance.

However, an essential condition for the above is that the steps in the assembly instructions are followed when assembling and installing the bed.

It is also necessary to ensure that the bed is set up in a manner that avoids distortion and stresses. Uneven installation on carpet edges and sloping floors will result in distortion.

### **6.2. Indoor climate**

Protect your bed from direct sunlight or very bright light, as no wood or fabric is completely fade-proof. High-energy neon or halogen light can also cause colour and material changes as well as bleaching.

Ideally, the humidity level in the room should be between 45% and 55%. At lower values, natural materials become brittle and synthetic fibres have a higher tendency become electrostatically charged and/or soiled. Higher humidity levels promote mould formation.

- Ventilate your bedroom regularly (short periods of full ventilation).
- Ensure sufficient air circulation on and under the mattress. We recommend shaking pillows and duvets regularly and, ideally, leaving the latter pulled back for a few hours – don't replace the covers until the bed has been well aired.
- In particular, if an additional mattress topper is used, this must be turned from time to time.

### **6.3. Signs of wear**

Even the high-quality surfaces, fabrics and covers that we use cannot stand up to all the stresses of everyday life. For example, the following may cause damage or leave traces:

- sharp, pointed objects, such as bracelets, jewellery, necklaces, hair clips, etc.
- pet claws
- chemical substances, such as hair dyes, hair gel, hair spray, make-up, creams and lotions, cosmetics
- hair grease and sweat, perspiration/exhalation from persons taking regular medication
- alcoholic liquids and fruit juice

Do not cut off loose threads, instead draw them to the inside. Once a thread is broken, this will create a hole in the cover.

Signs of wear of this type are not covered by the warranty and are not grounds for a claim.

### **6.4. Discoloration from contact with clothing or bed linen, pilling**

We can accept no liability for the colour fastness of other bedding and cover materials, whose colours may be transferred to the upholstered bed. This is not caused by a defect in the upholstery fabric, but rather by a defect in the clothing fabric or the bed linen used.

Pilling refers to the formation of small bobbles on fabric, generally caused by the friction of different fabrics rubbing together (non-genuine pilling), e.g. bed linen. The tendency to pilling can be increased by a humid atmosphere and/or electrostatic charge. This is a typical quality of these goods and not a defect. Pilling can be easily removed with a special fabric shaver.

### **6.5. Colour variation**

Along with colour variations resulting from production and finishing techniques (due to different stroke directions), the cover materials display the kind of colour variations that are typical for products of this kind. It is therefore also possible that their colours may differ slightly from those of display samples or previously delivered parts.

## **6.6. Finishing features**

According to the look of the upholstery, upholstered products are given either a tight-fitting or a looser finish. A looser finish means increased comfort since the user can sink gently into the upholstery. The desired look is created during use as a result of individual fold and ripple formation due to the somewhat looser covering, e.g. on mattresses covered in furnishing fabrics. After use, the surfaces can be smoothed again by working the material towards the outside, or by shaking.

Many soft materials are used in the upholstered area. Upholstery and sewing are manual crafts, therefore deviations from the specified dimensions are possible. The length and width of a mattresses may vary by  $\pm 1\%$ . Dimensional deviations may also be caused during storage and transport. Generally, the product will recover during use and regain its nominal dimensions.

## **6.7. Odours when new, changes during use**

New products always emit a material-specific odour. This is completely harmless and will disappear over time. Frequent ventilation will accelerate this process.

Every upholstered bed and every mattress will adjust to the occupant over time. The high-quality, fine layers of padding become compressed, creating an indentation in the surface of the mattress. Regular turning of the mattress, head to foot or right over, will allow the padding to recover.

Mattresses become a little softer with use, on average around 10–15%. These qualities and divergences from the specified firmness are typical of the goods and are not covered by the warranty.

## **6.8. Care and stain removal**

### **Regular care**

To remove dust, crumbs and surface dirt, lightly vacuum with a soft upholstery nozzle and then brush down with a soft brush.

Never vacuum mattress toppers or mattresses with fabric covers. Never use steam cleaners!

For mattresses with mattress covers or toppers, follow the instructions on the stitched-on care labels.

### **Imitation leather**

Carefully wipe down the cover with a clean, slightly damp chamois leather. Moisture will keep the material elastic. Treat stubborn stains with mild soapy water.

### **Stains**

Act quickly – the sooner you tackle a stain, the greater your chances of success. Depending on the circumstances, work from seam to seam, but always from the centre to the edge to avoid enlarging the stain. Use a clean, soft, white cloth and only apply light pressure when cleaning to avoid damaging the surface structure.

### **Water-soluble stains**

Blood, egg, faeces, urine	Treat with cold, distilled water. If necessary, re-treat with a shampoo solution or mild fabric detergent. Do not use hot water as the proteins will coagulate.
Coffee with milk, cocoa, indelible pencil, ballpoint pen, lipstick, mayonnaise, perfume, cream, sauces, soups, vomit	Treat with a lukewarm shampoo solution or mild fabric detergent and distilled water. If traces remain, treat with ethyl alcohol, cleaning solvent or stain remover after drying.
Spirits, beer, cola-based drinks, fruit juices, lemonade, coffee, tea	Do not allow the stain to dry, instead treat immediately with a lukewarm shampoo solution or mild fabric detergent and distilled water.

### **Non-water-soluble stains**

Butter, wax floor polish, paint, fat, varnish, nail varnish, furniture polish, oil, resin, charcoal, indelible pencil	Treat with solvents such as cleaning solvent, 75% isopropyl alcohol, ethyl alcohol or standard stain remover.
Chewing gum, plasticine	Use a standard freezing spray according to the instructions. With velour, there is a danger of damage to the surface.
Candle wax	Do not use an iron! Crumble the wax away as much as possible and

	<ul style="list-style-type: none"> <li>- carefully lift off (with velour, there is danger of damage to the surface).</li> <li>- If necessary, re-treat several times with cleaning solvent.</li> </ul>
Dried blood.	Moisten a white cloth with citric acid solution (1 level tablespoon to 100 ml cold water) and apply. Do not use on cotton, linen or viscose – acid will damage the fibres.

As a general rule:

- Carefully remove thick areas with a spoon. Never try to scrape or pull off dried-on stains.
- Soak a cloth in detergent or stain remover according to the recommendation.
- Cleaning agents must never be applied directly to the cover material!
- If foreign media are used, first apply them to a non-visible area to test the colour fastness of the material and check for changes to the finish.
- Never soak stains.  
Always use distilled water to avoid limescale deposits.
- For stains of uncertain origin, first proceed in the same way as for water-soluble stains. If this is not successful, apply the recommended treatment for non-water-soluble stains.
- After treatment, pat the cover dry with a dry white cloth, then brush down and allow to dry at room temperature. After drying, brush again with a soft brush.
- For persistent stains, seek advice from a specialist.
- Never use steam cleaners!


## 7. Conformity

The EOS Relax is manufactured in accordance with the ISO 9001:2008 standard and complies with the European Medical Devices Directive 93/42/EEC and all related harmonised standards:

EN 60601-1	Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.
EN 60601-1-2	Medical electrical equipment – Part 1-2: – Basic requirements for safety and essential performance. Accompanying standards: electromagnetic compatibility – requirements and tests.
EN 60601-2-52	Medical electrical equipment – Part 2-52 : Special requirements for the safety and essential features of medical beds.
EN ISO 14971	Risk management for medical equipment.
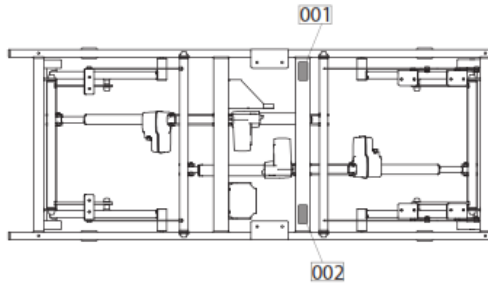
## 7.1. Explanation of the CE label and identification label


The labels are attached to the crossbar of the metal frame.

### 001 Identification label

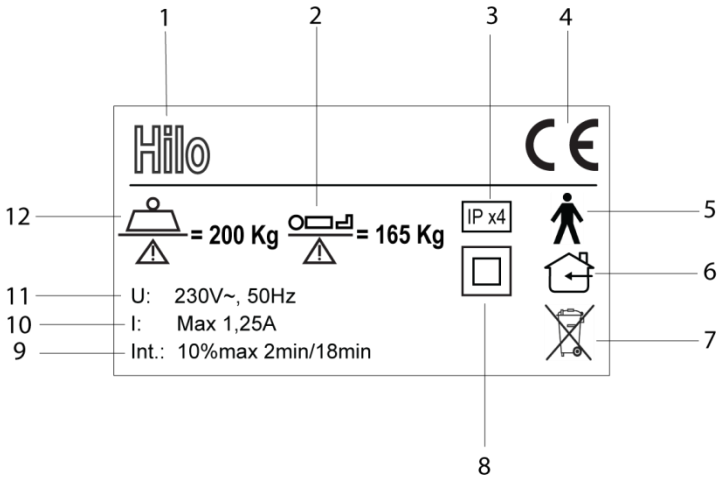002 CE label

- |   |  |    |                          |
|---|--|----|--------------------------|
| 1 | Item name  | 10 | Max. current consumption |
| 2 | Max. user weight   | 11 | Input voltage            |
| 3 | Protection type (spray-water from all directions)                                | 12 | Safe working load        |
| 4 | CE label in compliance with the Medical electrical equipment directive 93/42/EEC | 13 | Production date          |
| 5 | Type B equipment in compliance with EN 60601-1                                   | 14 | Manufacturer's address   |
| 6 | For indoor use only  | 15 | Manufacturer's website   |
| 7 | Warning: electronic waste  | 16 | Manufacturer             |
| 8 | Application class II   | 17 | Serial number            |
| 9 | Max. running time for motors is 2 minutes, then wait for 18 minutes              | 18 | Item name                |
|   |  | 19 | Item number              |

## 7.2. Declaration of conformity

### EC- DECLARATION OF CONFORMITY

Following the EC Directive concerning medical devices 93/42/EEC, annex VII.

I, the undersigned, agent of the following manufacturer:

**Haelvoet nv**

**Leon Bekaertstraat 8**

**8770 Ingelmunster**

**Belgium**

**Tel.:** **+32 (0) 51 48 66 95**

**Fax:** **+32 (0) 51 48 73 19**

**Email:** [info@haelvoet.com](mailto:info@haelvoet.com)

Declare hereby that the following product:

**Hilo**

**No.: 12376**

Medical device class I (non-invasive device)

when installed, maintained and used in accordance with the manual, the rules of good craftsmanship, and the intended purpose complies with all necessary safety requirements and other relevant provisions of annex I of:

#### Medical Devices directive 93/42/EEC

The following norms have been applied to indicate the conformity:

**EN 60601-1** Medical electrical equipment - Part 1 : General requirements for basic safety and essential performance.

**EN 60601-1-2** Medical electrical equipment - Part 1-2 : General requirements for basic safety and essential performance - Collateral standard : Electromagnetic compatibility - Requirements and tests.

**EN 60601-2-52** Medical electrical equipment - Part 2-52: Particular requirements for basic safety and essential performance of medical beds.

**EN ISO 14971** Medical devices - Application of risk management to medical devices.

The above-mentioned product has been designed, produced and checked in accordance with the quality management system of **ISO 9001:2008**.

**Ingelmunster, 22/08/2015**

**Signature:**

**Haelvoet Vincent**  
**Managing director**

## **8. Disposal and the environment**

Please separate the remaining waste in accordance with the official environmental regulations. Any electrical components such as motors, controllers and manual switches that are replaced must be treated as electronic waste. If you decide to stop using the bed and wish to dispose of it, dismantle the bed and dispose of it in accordance with the applicable environmental regulations.

This product contains reusable aluminium, steel, plastics and electrical components. To ensure optimal disposal at the end of the product's life cycle, separate all parts so that they can be recycled as base raw materials.

DOC AG is fully convinced of the need to protect the environment in order to preserve it for the generations to come. For this reason, we pay particular attention to the use of environmentally friendly materials and technologies in our development, innovation and production process.

This product has been manufactured using environmentally friendly materials. It contains no dangerous substances such as cadmium, mercury, asbestos, PCBs or CFCs.

This product's noise emissions meet the legal requirements for protection against unwanted noise and vibration.

This product contains lead-acid batteries (optional). Please dispose of the batteries at the designated and authorised places.

## Annex 1 – Maintenance and servicing checklist

<b>Customer:</b>					
<b>Address:</b>					
<b>Reason:</b>	<input type="checkbox"/> Check on delivery		<input type="checkbox"/> Periodic check		
<input type="checkbox"/> Other	<input type="checkbox"/> Check after repair or maintenance				
<b>Bed type:</b>	<input type="checkbox"/> Care bed	<input type="checkbox"/> Not a care bed	<input type="checkbox"/> Protection class: I	<input type="checkbox"/> Protection class: II	
<b>Model:</b>	<b>EOS Relax</b>		<b>ID no.:</b>		
<b>Installation:</b>					
<b>Manufacturer:</b>					
<b>Class:</b>	Class I non-invasive medical equipment				
<b>I. Visual check</b>			<b>OK</b>	<b>NOK</b>	<b>Fault description</b>
<b>Visual check: electronic components</b>					
Markings/labels	present				
Controller housing	Position correct, damage				
Motor housing	Position correct, damage				
Battery	Age				
Motor brackets	Bolts and fuses				
Manual switch	Cracks, damage				
Motor and manual switch cables	Check cable and attachment				
Controller connections	Properly connected, O-rings present, bayonet fitting				
Strain relief for power cable	Firmly fitted				
<b>Visual check: mechanical components</b>					
Markings/labels	present				
Up/down system	Damage, deformation				
Transport castors	Damage				
Welds/seams	Cracks, deformation				
Motor rods	Deformation, cracks, breaks				
Nuts and bolts					
<b>II. Electrical measurement in accordance with EN 62353</b>			<b>OK</b>	<b>NOK</b>	<b>Fault description</b>
<b>III. Function test for electronic components</b>			<b>OK</b>	<b>NOK</b>	<b>Fault description</b>
Motors and controller	Smooth running,				

	push rod greased			
Noises from motors/controller				
Battery	Performance			
Manual switch	Function of the buttons, noises, magnetic function, mounting			
Limit switches on motors	Automatic			
<b>Function test for mechanical components</b>				
All bed functions				
Hinges and visible pivot points	Smooth, easy running, no noise			
<b>Result</b>				
<b>All results are within the permitted specifications</b>	<input type="checkbox"/> yes <input type="checkbox"/> no		<b>Next inspection on:</b>	
<b>Check passed</b>	<input type="checkbox"/> yes <input type="checkbox"/> no			
<b>Check failed</b>	<input type="checkbox"/> Malfunction, do not use bed → Repair <input type="checkbox"/> Malfunction, do not use bed → Lock the bed and prevent further use <input type="checkbox"/> Bed no longer meets safety standards			
<b>Test sticker is present</b>	<input type="checkbox"/> yes <input type="checkbox"/> no			
<b>Additional comments</b>				
<b>Date:</b>	<b>Name:</b>		<b>Initials:</b>	

## Annex 2 – Symbols used

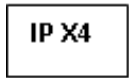
Safe working load


Safe user weight

Type B equipment in accordance with  
EN 60601-1

Electrical protection class

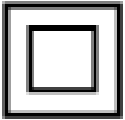Application class II

For indoor use only

Double insulated transformer

Thermal protection switch in the trans-  
former

Conformity label

Read the operating manual before using the product

Warning:  
failure to follow the instruction may result in accidents and serious injury

Failure to follow the instruction may result in damage to the bed and/or the environment

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